



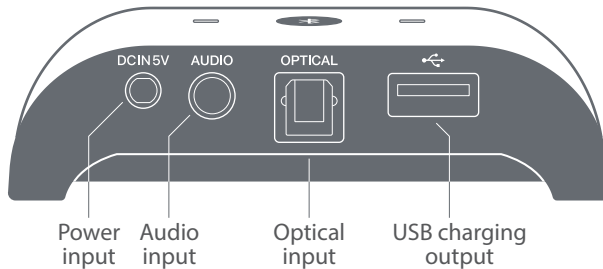

BE2022 Maxi Pro TV streamer

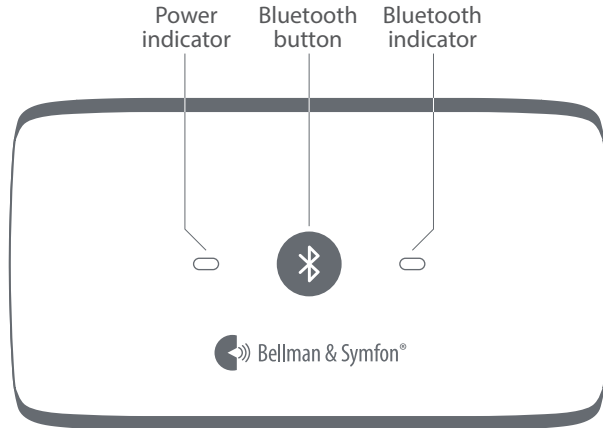

About the TV streamer

The Maxi Pro TV streamer enables you to hear the television at a volume that suits you, without affecting the volume for anyone else in the room. It streams the sound in full stereo via Bluetooth to your Maxi Pro personal amplifier.

What's in the box

Along with the TV streamer, the following items are included:

A mini-stereo cable



A mini-stereo-to-RCA adapter



A fiber optic cable



A DC-connector-to-USB cord




What you need to get started

To set up your TV streamer, you also need:

A TV with an RCA, headphone or optical audio port

A Maxi Pro personal amplifier

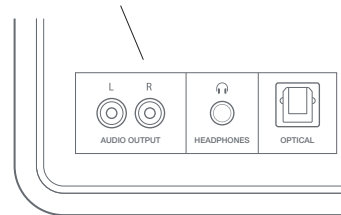

Setting up the TV streamer


Step 1: Connecting audio

There are several ways to connect the TV streamer to your TV. It depends on the type of audio ports your TV is equipped with.

Using the TV's RCA output

- 1 Connect one end of the mini-stereo cable to the audio input on the back of the TV streamer.
- 2 Connect the other end to the mini-stereo-to-RCA adapter.
- 3 Connect the RCA plugs to the audio output on your TV.

Note: You can also connect the mini-stereo cable directly to the headphone output on your TV. Please note that this may mute the speakers on some TV models.

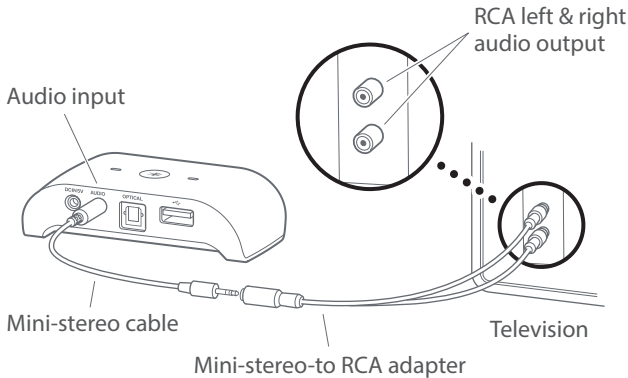


Using the TV's optical output

If your TV features an optical output, you can use the fiber optic cable to connect the TV streamer to your TV. Here is how you connect it:

- 1 Connect one end of the cable to the optical input on the back of the TV streamer.
- 2 Connect the other end to the optical output on your TV.


Fiber optic cable

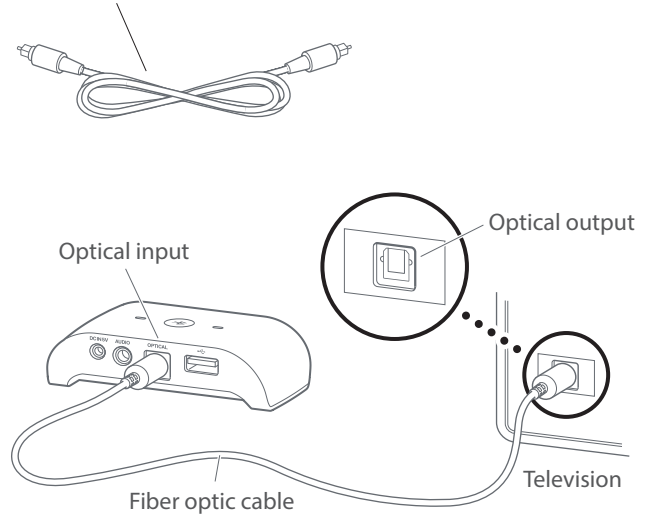

Step 2: Connecting power

- 1 Connect the DC end of the power cord to the power input on the back of the TV streamer.
- 2 Connect the USB end to the Maxi power adapter.
- 3 Fit the adapter plug to the power adapter and connect it to a wall outlet. The power indicator lights up in green.

Note: You can also power it by connecting the USB end of the power cord to a TV USB port. This however might delay the TV sound in your headphones during reconnection.

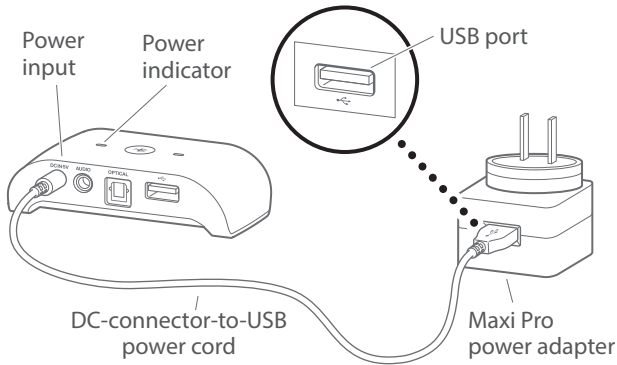

Step 3: Pairing

Note: If the TV streamer came with your Maxi Pro, you don't need to pair the units.

- 1 **On the TV streamer:** Press and hold the Bluetooth button until the Bluetooth indicator starts to blink in blue.
- 2 **On your Maxi Pro:** Press the microphone selector. Press and hold the Bluetooth selector until the Bluetooth indicator starts to blink in blue.
- 3 Both Bluetooth indicators will emit a steady blue light to show that the pairing was successful.

Using your TV streamer


As long as your TV streamer is connected to power, it will remain on and automatically reconnect to your Maxi Pro when it returns within Bluetooth range. Both Bluetooth indicators will light up in blue when the units are connected.

Note: If you power the TV streamer from your TV, it will start automatically when you turn on the TV and reconnect with your Maxi within 10 seconds.

Charging your Maxi Pro

Here is how you charge your Maxi from the TV streamer:

- 1 Connect the USB end of the Maxi charging cable to the USB charging output on the back of the TV streamer.
- 2 Connect the other end to the Maxi charger jack.
- 3 The Maxi battery indicator blinks in green during charging.

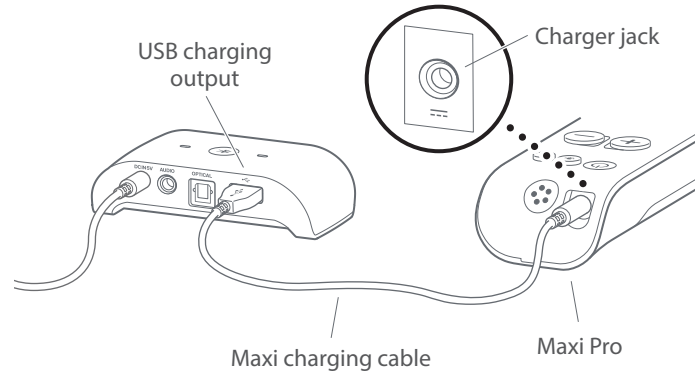

Troubleshooting

If	Try this
Nothing happens when I press the TV streamer Bluetooth button.	<ul style="list-style-type: none">• Make sure that the TV streamer is connected to power, see Connecting power.
I don't hear the sound of the TV in my headphones.	<ul style="list-style-type: none">• Make sure Maxi Pro is within range and paired with your TV streamer, see Pairing.
The TV sound disappears when I connect the mini-stereo cable to the headphones output.	<ul style="list-style-type: none">• This is a feature on some TV models. Use the RCA or optical output on your TV instead, see Connecting audio.

Care and maintenance

Operate the product in a dry environment where the temperature is always between 0° and 35° C.

Clean the product with a soft, lint-free cloth. Avoid getting moisture in openings. Don't use window cleaners, household cleaners, solvents, alcohol, ammonia, or abrasives.

If the product appears to be damaged or doesn't function properly, follow the instructions in this leaflet. If the product still doesn't function as intended, contact your local dealer for information on service and warranty.

European office

Bellman & Symfon AB
Sodra Langebergsgatan 30
436 32 Askim, Sweden
Phone +46 31 68 28 20
E-mail info@bellman.com

