
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning

**BE1432 VISIT MOBILE PHONE TRANSMITTER**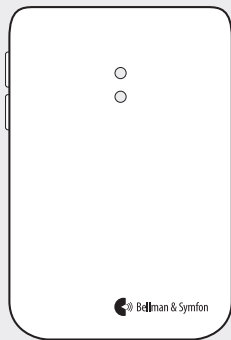

## Visit mobile phone transmitter

- 
- 03 Medical device information

---

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SE

# Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

## About the BE1432 Visit mobile phone transmitter

### Intended purpose

This device is part of the Visit alerting system. The intended purpose of the system is to alert deaf and hard of hearing people of important signals in their home. The transmitter detects when the landline and mobile phone rings (or receives a message) and sends a signal to the Visit receiver that alerts the user with sound, flashes or vibrations. The transmitter is to be used together with the **Visit** smart device application.

### Intended user

The intended user is a person with mild to severe hearing loss or deafness.

### Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

### Principle of operation

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations. In order to use the system, you need at least one transmitter and one receiver.

The Visit alerting system will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

## Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

## Hazard warnings

-  Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
-  Make sure to keep this device out of reach of children under 3 years.
-  Note that alarms and notifications can be missed if batteries run out.
-  This product uses Alkaline batteries as power backup. Do not expose the batteries to fire or to direct sunlight. Only use 1.5 V AA alkaline batteries.
-  This device is designed for indoor use only. Do not expose the device to moisture.
-  Protect the device from shocks during storage and transport.
-  This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the user guide, may cause harmful interference to radio communication.
-  Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
-  Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.

-  Protect cables from any potential source of damage.
-  Do not make any changes or modifications to this device. Use only power adapters and accessories

-  Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

## Information on product safety

-  Failure to follow these instructions could result in damage to the device and void the warranty.
-  Do not expose batteries to fire or to direct sunlight.
-  Do not drop your device. Dropping onto a hard surface can damage it
-  Do not use the device in areas where electronic equipment is prohibited.
-  The device may only be repaired by an authorized service center.
-  If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
-  If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit **bellman.com** for contact info.

 This device should not be used on aircrafts unless specifically permitted by flight personnel.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the product gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia, abrasives or compressed air.

## Service and support

If the product appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused.

This guarantee excludes incidental and consequential damage. Further the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary from state to state. Some states or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty last so the above limitation may not apply to you.

This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Configuration options

This transmitter can be configured with the following Visit receivers and accessories:

### Compatible Visit receivers

- BE1450 Visit portable receiver
- BE1441 Visit flash receiver
- BE1442 Visit flash receiver
- BE1580 Visit alarm clock receiver
- BE1470 Visit pager receiver
- BE1560 Visit wrist receiver

### Available accessories

- BE9026 Contact mat
- BE9023 Magnetic switch
- BE9253 External trig cable

For detailed product information, see the corresponding user manual.

## ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.  
SS-EN ISO 9001 Certification No.: CN19/42071, SS-EN ISO 13485 Certification No.: CN19/42070

### Certification Body

SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

---

Radio Equipment Directive (RED)

RoHS Directive

---

Medical Device Regulation (MDR)

REACH Regulation

---

Electromagnetic Compatibility Directive (EMC)

EC General Product Safety Directive

---

Waste Electrical & Electronic Equipment (WEEE)

EC Battery Directive

---

# Regulatory symbols

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instruction guide and this leaflet.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage:  $-10^{\circ}$  to  $50^{\circ}$  C,  $14^{\circ}$  –  $122^{\circ}$  F  
Temperature during operation:  $0^{\circ}$  to  $35^{\circ}$  C,  $32^{\circ}$  to  $95^{\circ}$  F

Humidity during transportation and storage:  $<90\%$ , non-condensing  
Humidity during operation:  $15\%$  –  $90\%$ , non-condensing

Atmospheric pressure during operation, transportation and storage:  
 $700$  hpa to  $1060$  hpa

---

**Operating conditions** This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.

---

 With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

---

 This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

---

 The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Bellman & Symfon is under license. Other trademarks and trade names are those of their respective owners.

---

## Technical specifications

**Dimensions and weight** 100 × 65 × 27 mm, 4.0 × 2.6 × 1.1"  
Weight: 138 g, 4.9 oz. including battery and telephone cable

**Power and battery** Mains power: 5 V DC / 1000 mA  
Backup batteries: 2 × 1.5 V AA Alkaline

**Connectivity** RJ11 input to analogue telephone network, 3.5 mm ext. trig input, Power supply jack, Bluetooth Low Energy (BLE)

**Frequency and coverage** ISM frequency: 868.30 MHz, Coverage: Up to 250 m, depending on the characteristics of the building.  
Bluetooth freq: 2402 – 2480 MHz, Coverage: Up to 50 m

**Compatibility** iOS11 and later / Android 4.4 and later  
iPhone 6S and later / Modern Android devices

**In the box**

- BE1432 Mobile phone transmitter
- Power supply and power cable
- 2 × 1.5V AA alkaline batteries
- Screw and plug
- Telephone splitter
- Telephone cord

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TM and © 2020 Bellman & Symfon AB. All rights reserved.
Bellman & Symfon Group AB