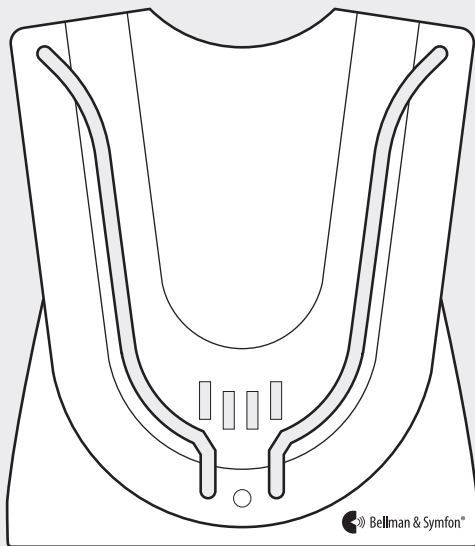
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning

## Pager charger

- 
- 3 Medical device information
- 
- 11 Informationen zu Medizinprodukten
- 
- 19 Lääketieteellisiä laitteita koskevat tiedot
- 
- 27 Informatie over medische hulpmiddelen
- 
- 35 Informasjon om medisinsk utstyr
- 
- 43 Information om medicinteknisk utrustning
- 

EN

DE

FI

NL

NO

SE

# Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

## About the BE1260 Pager charger

### Intended purpose

The intended purpose of this pager charger accessory is to recharge the BE1470 Visit pager receiver battery and to power up to two optional bed shakers.

### Principle of operation

The pager charger is typically placed on the bedside table and used during the night. It can be equipped with up to two BE1270 Bed shaker accessories that awakes the user with vibrations under the pillow or mattress if any of the Visit transmitters are activated.

### Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

### Intended user

The intended user is a person with mild to severe hearing loss or deafness.

 This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

# Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

## Hazard warnings

-  Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.
-  Make sure to keep this device out of reach of children under 3 years.
-  Be aware that alarms and notifications can be missed if batteries run out.
-  This device should not be used as the only means of security in life-threatening situations.
-  Use only power adapters and battery types that are specified in this leaflet.
-  **Only** use rechargeable 600mAh - 650mAh NiMH batteries for the pager when you are using the pager charger. Non-rechargeable batteries will start to leak if the pager is placed in the charger, which might damage the electronics or cause an explosion. This is not covered by warranty.
-  Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.
-  Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.
-  This device is designed for indoor use only. Do not expose the device to moisture.
-  Protect the device from shocks during storage and transport.
-  Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.
-  Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

## Information on product safety

-  Failure to follow these instructions could result in damage to the device and void the warranty.
-  Do not drop your device. Dropping onto a hard surface can damage it.
-  Do not use the device in areas where electronic equipment is prohibited.
-  The device may only be repaired by an authorized service center.
-  If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.
-  Do not expose batteries to fire or direct sunlight.
-  If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](https://bellman.com) for contact info.
-  Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.
-  Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

## Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for six (6) months from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

# Regulatory symbols

 With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

 This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

 This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

 This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

 This symbol indicates that the user should consult the instruction guide and this leaflet.

 This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.

 This symbol indicates important information for handling and product safety.

 Temperature during transport and storage: -10° to 50° C, 14° to 122° F  
Temperature during operation: 0° to 35° C, 32° to 95° F

 Humidity during transportation and storage: <90%, non-condensing  
Humidity during operation: 15% to 90%, non-condensing

 Atmospheric pressure during operation, transportation and storage: 700 hpa to 1060 hpa

**Operating conditions** This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.

 With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection.

 This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

## Technical specifications

<b>Dimensions</b>	78 × 88 × 43 mm, 3.1 × 3.5 × 1.7"
<b>Weight</b>	200 g, 7 oz. incl. batteries and power adapter
<b>Mains power</b>	7.5 VDC / 1500 mA
<b>Battery power</b>	2 × 1.2 V NiMH rechargeable batteries. <b>Note:</b> The backup batteries must be changed at a service center
<b>Pager charging time</b>	Normal charging time: ~ 6 h With depleted backup batteries: ~24h
<b>Vibrator power</b>	2.0 – 4.0 VDC

**Charging LED** Green light: The pager is charging  
No light: The pager is fully charged or the power adapter is not connected to mains power.

**In the box**

- BE1260 Pager charger with pre-mounted backup batteries
- External power supply
- 1.2 V NiHM rechargeable battery intended for the pager
- Supporting legs
- Screws and plugs

## Configurations

This BE1260 Pager charger is an accessory to the BE1470 Visit pager receiver. It can be configured with the following Bellman & Symfon original accessories:

### Available accessories

- BE1270 Bed shaker  
Connects up to two bed shakers

 For detailed product information, see the corresponding user manual.

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

Medical Device Regulation (MDR)	REACH Regulation
Radio Equipment Directive (RED)	EC General Product Safety Directive
Electromagnetic Compatibility Directive (EMC)	Waste Electrical & Electronic Equipment (WEEE)
RoHS Directive	EC Battery Directive

### ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071

SS-EN ISO 13485 Certification Number: CN19/42070

### Certification Body

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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TM and © 2021 Bellman & Symfon AB. All rights reserved.
SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN Storbritannien
Bellman & Symfon Group AB