
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning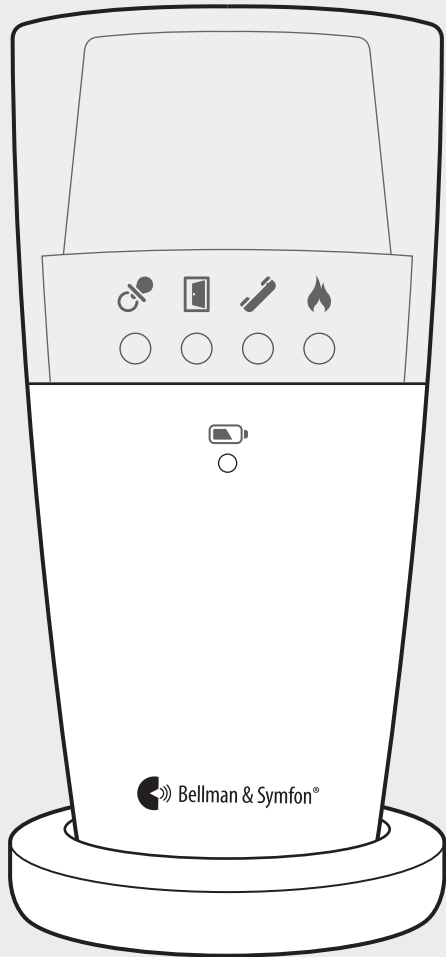

## Visit flash receiver

---

03 Medical device information

---

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SE

# Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

## About the BE1441 and BE1442 Visit flash receiver

### Intended purpose

This flash receiver is part of the Visit alerting system. The intended purpose of the system is to alert deaf and hard of hearing people of important signals in their home. Depending on the system setup, the flash receiver alerts the user with bright flashing light when for instance the doorbell or telephone rings, the baby needs attention, or the smoke alarm goes off.

### Intended user group

The intended user group consists of people of all ages with mild to severe hearing loss or deafness that need audio, visual or sensory amplification.

### Intended user

The intended user is a person with mild to severe hearing loss or deafness.

### Principle of operation

The Visit alerting system consists of a set of wirelessly connected transmitters and receivers that are located across the home. When a transmitter detects an activity, it signals the Visit receiver that alerts the user with sound, flashes or vibrations, depending on the type of receiver. In order to use the system, you need at least one transmitter and one receiver.

 The Visit alerting system will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

## Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

### Hazard warnings

 Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.

 Keep this device out of reach of children under 3 years.

 Be aware that alarms and notifications can be missed if batteries run out.

 Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

 Flashes can trigger epileptic seizures for people with photosensitive epilepsy (PSE).

 Use only power adapters and battery types that are specified in this leaflet.

 Protect cables from any potential source of damage.

 Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.

 This device is designed for indoor use only. Do not expose the device to moisture.

 Protect the device from shocks during storage and transport.

 Do not make any changes or modifications to this device. Use only original Bellman & Symfon accessories to avoid any electrical shock.

 Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

 Use a 26 AWG or greater wire diameter for the flash receiver telephone jack.

## Information on product safety

 Failure to follow these instructions could result in damage to the device and void the warranty.

 Do not expose batteries to fire or to direct sunlight.

 Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.

 Rechargeable batteries should only be changed by Bellman & Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.

 Do not drop your device. Dropping onto a hard surface can damage it

 Do not use the device in areas where electronic equipment is prohibited.

 The device may only be repaired by an authorized service center.

 If a serious incident occurs in relation to this device, contact the manufacturer and relevant authority.

 If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](http://bellman.com) for contact information.

 The programming pin on the back of the flash receiver is for authorized service personnel only. Unauthorized use may damage the device.

 Without permission granted by the NCC, no company, enterprise or user is allowed to change the frequency, enhance the transmitting power or alter the original characteristics or performance on an approved low power radio frequency device.

 The low power radio frequency devices shall not influence aircraft security and interfere legal communications; If found, the user shall cease operating immediately until no interference is achieved. The said legal communications means radio communications is operated in compliance with the Telecommunications Act. The low power radio frequency devices must be susceptible with the interference from legal communications or ISM radio wave radiated devices.

## Configuration options

The BE1441 and BE1442 flash receiver model can be configured with the following Visit transmitters and Bellman & Symfon original accessories:

### Compatible Visit transmitters

- BE1411 Visit door transmitter
- BE1420 Visit pushbutton transmitter
- BE1431 Visit telephone transmitter
- BE1432 Visit mobile phone transmitter
- BE1481 Visit smoke alarm transmitter
- BE1551 Visit smoke alarm transmitter
- BE1555 Visit CO alarm transmitter
- BE1491 Visit baby monitor

### Available accessories

- BE9075 Wall mount bracket
- BE1270 Bed shaker
- BE9105 Telephone cable

 For detailed product information, see the corresponding user manual.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

## Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product (excluding the battery) for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Regulatory symbols

With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

This symbol indicates that the user should consult the instruction guide and this leaflet.

This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.

This symbol indicates important information for handling and product safety.

Temperature during transport and storage:  $-10^{\circ}$  to  $50^{\circ}$  C,  $14^{\circ}$  to  $122^{\circ}$  F  
Temperature during operation:  $0^{\circ}$  to  $35^{\circ}$  C,  $32^{\circ}$  to  $95^{\circ}$  F

Humidity during transportation and storage: <90%, non-condensing  
Humidity during operation: 15% to 90%, non-condensing

Atmospheric pressure during operation, transportation and storage: 700hpa to 1060hpa

This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.

With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

# Technical specifications

<b>Dimensions and weight</b>	BE1441: 70 × 140 mm, 2.7 × 5.5" (Ø × H) BE1442: 78 × 155 mm, 3.1 × 6.1" (Ø × H) Weight: 308 g, 10.8 oz. (both models)
<b>Mains power</b>	7.5 V DC / 1500 mA Only use the supplied power adapter. This product is also designed for IT power distribution systems with phase to phase voltage of up to 230V. The outlet shall be installed near the equipment and shall be easily accessible.
<b>Power consumption</b>	Active: 900 mA, idle position: 10 mA
<b>BE1442 battery backup</b>	Battery type: 4 × 1.2 V Level 600/900 mAh Operating time: ~ 48 h when fully charged
<b>Connections</b>	<ul style="list-style-type: none"><li>▪ 2.5 mm external trigger jack</li><li>▪ 3.5 mm bed shaker outlet, vibration power: 2.0 – 4.0 VDC (max. 500 mA)</li><li>▪ RJ-11 telephone modular jack, 26 – 120 V RMS, 15 – 100 Hz</li><li>▪ Barrel jack for the external power adapter</li><li>▪ Programming pin for service purposes. Output: 3.3 V/max. 150mA.</li></ul>
<b>Flash</b>	Built-in ~30 Candela Xenon light Warning! Flashes can cause epileptic seizures
<b>Radio frequency</b>	314.91 MHz, 433.92 MHz or 868.30 MHz depending on region
<b>Coverage</b>	50 – 250 m, 55 – 273 yards depending on the radio frequency and the characteristics of the building. Coverage is reduced if walls and large objects block the signal.
<b>In the box</b>	<ul style="list-style-type: none"><li>▪ BE1441 Visit flash receiver or BE1442 Visit flash receiver with battery backup</li><li>▪ 7.5 V DC / 1500 mA power adapter</li><li>▪ 4 × 1.2 V AAA NiMH batteries (BE1442 model only)</li></ul>

## ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071

SS-EN ISO 13485 Certification Number: CN19/42070

### Certification Body

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

# Compliance information

## Europe

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

---

Radio Equipment Directive (RED)

---

Medical Device Regulation (MDR)

---

Electromagnetic Compatibility Directive (EMC)

---

Restriction of Hazardous Substances Directive (RoHS)

---

REACH Regulation

---

EC General Product Safety Directive.

---

Waste Electrical & Electronic Equipment, WEEE

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TM and © 2020 Bellman & Symfon AB. All rights reserved.
Bellman & Symfon Group AB