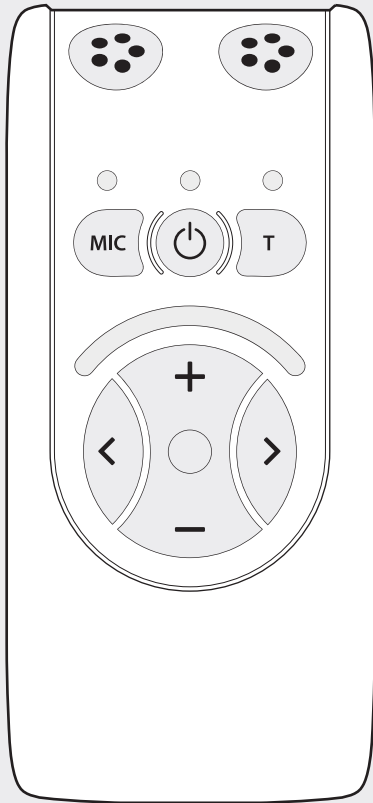
- EN** Medical device information
- DE** Informationen zu Medizinprodukten
- FI** Lääketieteellisiä laitteita koskevat tiedot
- NL** Informatie over medische hulpmiddelen
- NO** Informasjon om medisinsk utstyr
- SE** Information om medicinteknisk utrustning

## Mino personal amplifier

- 
- 03 Medical device information

---

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EN  
DE  
FI  
NL  
NO  
SE

## Read this first

Thank you for choosing a product from Bellman & Symfon – the world leader in alerting systems based in Gothenburg, Sweden. This leaflet contains important medical device information. Please read it carefully to make sure that you understand and get the best out of your Bellman & Symfon product. For more information about features and benefits, contact your hearing care professional.

### About the BE2030 Mino personal amplifier

#### Intended purpose

The intended purpose of this assistive listening device is to amplify the volume and improve the speech intelligibility during conversations and TV-listening. It can also be used with other sound sources.

#### Principle of operation

The device is typically placed on a table or hung around the neck. It picks up the surrounding sound via an internal microphone, clarifies the speech and suppresses the background noise. The user can listen with headphones, earphones or via hearing aids (neckloop accessory required).

#### Intended user group

The intended user group consists of people of all ages experiencing mild to severe hearing loss that need sound amplification in different situations.

#### Intended user

The intended user is a person with mild to severe hearing loss that need sound amplification.

 This device will not restore normal hearing and will not prevent or improve a hearing impairment or deafness resulting from organic conditions.

## Important safety information

This section contains important information about safety, handling and operating conditions. Keep this leaflet for future use. If you are just installing the device, this leaflet must be given to the householder.

### Hazard warnings

 Failure to follow these safety instructions could result in fire, electric shock, or other injury or damage to the device or other property.

 Keep this device out of reach of children under 3 years.

 Do not use or store this device near any heat sources such as naked flames, radiators, ovens or other devices that produce heat.

 Protect cables from any potential source of damage.

 This device has a very powerful amplifier and the sound level can cause discomfort or, in serious cases, endanger hearing if the product is used carelessly. Always turn the volume down to a low level before putting on the headphones!

 Protect the device from shocks during storage and transport.

 Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

 Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

 Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm, 12" to any part of this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.

 The power adapter must not be exposed to water drops or splashes and objects containing liquids, such as vases must not be placed on the unit. To disconnect the power fully, the adapter plug must be disconnected from the mains socket. The adapter plug must not be blocked and must be easy to access for the intended use. Rating information is shown on the rear.

 Do not block any of the ventilation openings. Install the device in accordance with the manufacturer's instructions.

 Do not defeat the safety purpose of the polarized or grounding-type plug. A polarized plug has two blades with one wider than the other. A grounding type plug has two blades and a third grounding prong. The wide blade or the third prong are provided for your safety. When the provided plug does not fit into your outlet, consult an electrician for replacement of the obsolete outlet.

 Protect the power cord from being walked on or pinched particularly at plugs, convenience receptacles, and the point where they exit from the unit.

 Unplug this apparatus during lightning storms or when unused for long periods of time.

 Do not dismantle the device; there is a risk of electric shock. Tampering with or dismantling the device will void warranty.

 This device is designed for indoor use only. Do not expose the device to moisture.

 Batteries are toxic. Do not swallow them! Keep out of reach of children and pets. If they are swallowed, consult your physician immediately!

### Information on product safety

 Failure to follow these instructions could result in damage to the device and void the warranty.

 Do not drop your device. Dropping onto a hard surface can damage it.

 The device may only be repaired by an authorized service center.

 If a serious incident occurs in relation to this device, contact the

manufacturer and relevant authority.

 If you encounter other problems with your device, contact the point of purchase, your local Bellman & Symfon office or the manufacturer. Visit [bellman.com](http://bellman.com) for contact info.

 This device should not be used on aircrafts unless specifically permitted by flight personnel.

 Rechargeable batteries should be charged fully before using the device for the first time. Only charge batteries in a temperature between 0°– 35°C, 32°– 95° F.

 Rechargeable batteries should only be changed by Bellman &

Symfon or an authorized service center. There is a risk of explosion if the batteries are not replaced correctly.

 Do not use the device in areas where electronic equipment is prohibited.

## Configuration options

This device can be configured with the following original accessories:

### Available accessories

- BE9136 External microphone
- BE9159 Neck loop with 90 cm (3") cable
- BE9161 Neck loop with 15 cm (6") cable
- BE9126 Cable kit with 5 m, 16.4 ft.
- BE9122 Headphones
- BE9124 In-ear earphones
- BE9147 Travel case

 For detailed product information, see the corresponding user manual.

## Operating conditions

Operate the device in a dry environment within the temperature and humidity limits stated in this leaflet. If the device gets wet or is exposed to moisture, it should no longer be regarded as reliable and should therefore be replaced.

## Cleaning

Disconnect all cables before you clean your device. Use a soft, lint-free cloth. Avoid getting moisture in openings. Do not use household cleaners, aerosol sprays, solvents, alcohol, ammonia or abrasives. This device does not require sterilization.

## Service and support

If the device appears to be damaged or doesn't function properly, follow the instructions in the user guide and this leaflet. If the product still doesn't function as intended, contact your local hearing care professional for information on service and warranty.

## Warranty conditions

Bellman & Symfon guarantees this product for two (2) years from date of purchase against any defects that are due to faulty materials or workmanship. This guarantee only applies to normal conditions of use and service, and does not include damage resulting from accident, neglect, misuse, unauthorized dismantling, or contamination howsoever caused. This guarantee excludes incidental and consequential damage. Furthermore, the warranty does not cover Acts of God, such as fire, flood, hurricanes and tornadoes. This warranty gives you specific legal rights and you may also have other rights that vary with territory. Some countries or jurisdictions do not allow the limitation or exclusion of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above limitation may not apply to you. This guarantee is in addition to your statutory rights as a consumer. The above warranty may not be altered except in writing signed by both parties hereto.

## Regulatory symbols

 With this symbol, Bellman & Symfon confirms that the product meets the Medical Device Regulation EU 2017/745.

 This symbol indicates the manufacturer's serial number so that a specific medical device can be identified. It's available on the product and gift box.

 This symbol indicates the manufacturer's catalogue number so that the medical device can be identified. It's available on the product and gift box.

 This symbol indicates the medical device manufacturer, as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.

 This symbol indicates that the user should consult the instruction guide and this leaflet.

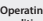
 This symbol indicates that it is important for the user to pay attention to the relevant warning notices in the user guides.

 This symbol indicates important information for handling and product safety.

 Temperature during transport and storage: -10° to 50° C, 14° to 122° F  
Temperature during operation: 0° to 35° C, 32° to 95° F

 Humidity during transportation and storage: <90%, non-condensing  
Humidity during operation: 15% to 90%, non-condensing

 Atmospheric pressure during operation, transportation and storage: 700hpa to 1060hpa

 This device is designed such that it functions without problems or restrictions if used as intended, unless otherwise noted in the user guide or this leaflet.

 With this CE symbol, Bellman & Symfon confirms that the product meets EU standards for health, safety, and environmental protection as well as the Radio Equipment Directive 2014/53/EU.

 This symbol indicates that the product shall not be treated as household waste. Please hand over your old or unused product to the applicable collection point for the recycling of electrical and electronic equipment or bring your old product to your hearing care professional for appropriate disposal. By ensuring this product is disposed of correctly, you will help prevent potential negative effects on the environment and human health.

# Technical specifications

## Power supply

**Battery power** Built-in rechargeable li-ion battery 3.7 V, 860 mAh

**Operating time** BE9124 Earphones – up to 30 hours  
BE9159 Neck Loop – up to 10 hours

**Charging device** 5V 1A DC adapter

**Charging time** ~ 4 hours

## Input signals

**Microphones** Built-in omnidirectional and directional

**Telecoil** Built-in telecoil angled at 45°

**Line input** 2.5 mm stereo jack, Max input: 0 dBV. Automatic selection between line input and external microphone.

## Output signals

**Headphone socket** 3.5 mm stereo jack, Output: 100 mW at 16 Ω load

**Distortion** BE9122 Headphones: 0.6% THD  
BE9124 Earphones: 0.08% THD

**Dynamic range** 90 dB SNR (electrical)

**Frequency range** 40 – 10,000 Hz (electrical)

**Max. output level** BE9122 Headphones: 124.3 dB SPL @ 2.5 kHz  
BE9124 Earphones: 132.0 dB SPL @ 5 kHz

**Amplification, full volume** BE9122 Headphones: 42.5 dB  
(1 kHz, 60 dB SPL input)  
BE9124 Earphones: 55.0 dB

**Amplification, full volume** BE9122 Headphones: 46.5 dB  
(60 dB SPL input HFA)  
BE9124 Earphones: 51.5 dB

## Features

**Volume control** 9 increments of 5 dB

**Tone control** +/- 10 dB in 9 increments, cut-off frequency 1.6 kHz

**On/Off button** Clear, countersunk pushbutton

**Mic and T-position** Separate pushbuttons

**Combined sound sources** For mixing external sound and sound from the internal microphone

**L/R balance setting** 0 – 20 dB reduction in increments of 5 dB

**Max. volume setting** 0 – 21 dB reduction in increments of 3 dB

**Bass reduction setting** Lowering of frequencies under 400 Hz by 10 dB

**Dynamic compressor** 10 channels  
Dynamic amplification 0-35 dB

Compression: 3:1

Expansion: 1:1.5

Attack time: 20 ms <3kHz, 25 ms ≥3kHz

Recovery time: 110 ms <3kHz, 200 ms ≥3kHz

**Dynamic noise reduction** 10 channel adaptive noise reduction

**Feedback handling** Adaptive feedback reduction

**Digital signal processing** 22,050 kHz sampling frequency  
16-bit resolution in stereo

**Additional features** Stereo/Mono setting  
Factory reset feature  
Casing designed to reduce contact noise

## Additional information

**Dimensions and weight** 48 × 99 × 22 mm, 1.9 × 3.9 × 0.9" (b × h × d)  
Weight: 75 g, 2.6 oz.

**In the box**

- BBE2030 Personal amplifier
- USB universal charger
- Mains plug
- Charger cable
- Neck strap with safety clasp
- Listening accessory (varies with territory)

## Compliance information

Hereby Bellman & Symfon declares that, in Europe, this product is in compliance with the essential requirements of the Medical Device Regulation EU 2017/745 as well as the directives and regulations listed below. The full text of the declaration of conformity can be obtained from Bellman & Symfon or your local Bellman & Symfon representative. Visit [bellman.com](http://bellman.com) for contact information.

- Medical Device Regulation (MDR)
- Electromagnetic Compatibility Directive (EMC)
- Restriction of Hazardous Substances Directive (RoHS)
- REACH Regulation
- EC General Product Safety Directive
- Waste Electrical & Electronic Equipment (WEEE)
- EC Battery Directive

## ISO Certification of legal manufacturer

Bellman is certified in accordance with SS-EN ISO 9001 and SS-EN ISO 13485.

SS-EN ISO 9001 Certification Number: CN19/42071

SS-EN ISO 13485 Certification Number: CN19/42070

### Certification Body

SGS United Kingdom Ltd

Rossmore Business Park Ellesmere Port Cheshire CH65 3EN UK

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TM and © 2020 Bellman & Symfon AB. All rights reserved.
SGS United Kingdom Ltd, Rossmore Business Park Ellesmere Port Cheshire CH65 3EN
Bellman & Symfon Group AB